Prophy Brushes

Description

Tooth cleaning brushes are designed to remove dental plaque and to polish tooth surfaces using prophylactic paste. The purpose is to reach esthetic clean surfaces. Different shapes are available for specific applications (front/occlusal and interproximal polishing). All rotating brushes are mounted on a metal stem designed to fit contra-angle, screw-in and snap-on mounting.

Application

Only use prophylactic brushes together with a prophylactic paste. Place a small quantity of paste on the brush or directly on the tooth. Start polishing the tooth surface with rotating or oscillating movements. Soft brushes also allow cleaning beneath the gingival margin.

Properties

Observe an optimal motor speed of 5.000 rpm. Do not use pressure. All brushes are for single use and are supplied non-sterile. Non-autoclavable.



Shelf life

60 Months



Form of delivery

Cup Brushes, 4 mm, mounted on CA cups

REF 260.21 with natural bristles, 12 pieces

REF 260.230 with natural bristles, 100 pieces

REF 260.2N white, with nylon filament, 12 pieces

REF 260.230N white, with nylon filament, 100 pieces

REF 260.240N blue, with nylon filament, 100 pieces

REF 260.260N white soft, with nylon filament, 100 pieces

REF 260.250N pink soft, with nylon filament, 100 pieces

Accessories

REF 210 mandrel with screw shank to be mounted on angled handpiece

REF 480 ProfiGuard RDA 250

REF 481 ProfiPolish RDA 40

REF 486 ProphyRepair for cleaning

and polishing, final RDA 34





Classification

Medical Device, class IIa, rule 5 CE-Certificate: G1 17 09 49628 010,

TÜV SÜD Product Service GmbH, München

Manufacturer

Alfred Becht GmbH, Carl-Zeiss-Str. 16, D-77656 Offenburg, Tel.: +49 781 60586-0,

Fax.: +49 781 60586-40, www.becht-online.de

