

## Prophy Repair

Prophylactic paste

### Description

All-in-one prophylactic paste for cleaning, polishing and remineralization of vital teeth. Progressive prophylactic paste. It contains silicates that change during application from coarse (cleaning) to fine (polishing). The significant hydroxylapatite content strengthens the tooth enamel and completes a professional treatment.

### Application

After removal of dental calculus place ProphyRepair into a dappen dish or other suitable container. Use a small quantity on a rubber polisher or polishing brush and apply to the area of treatment. Only start handpiece if polisher is in contact with the tooth surface. Polish at low speed and with little pressure. Allow polisher to reach areas below the gingiva. Repeat the treatment with fresh paste until all surfaces are cleaned and polished. Use Becht dental floss to apply the paste to the interproximal spaces. Finish by rinsing thoroughly.

### Properties

- white, RDA 34 (final polishing)
- water soluble
- only foams up slightly
- gluten and phthalate-free
- mild mint flavour
- 100 % vegan

### Composition

- Amorphous Siliciumdioxide
- Polyethylenglykol
- Paraben
- Hydroxylapatite (0,5%)
- flavor-agents



### Form of delivery

REF 486 Tube with 95 g / 75 ml

### Shelf life

48 months


### Classification

Medical device, class I, Rule 5,

### Accessories

Cup Brushes, 4 mm, mounted on CA cups:

- REF 260.21 white, with natural bristles
- REF 260.2N white, with nylon filament
- REF 260.240N blue, with nylon filament
- REF 260.250N soft, with nylon filament, pir
- REF 260.260N soft, with nylon filament, white

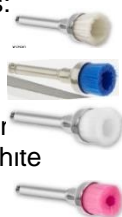

Prophylaxis Cups, latex-free

- REF 200.1, 200.2, 200.3 Screw-in cups
- REF 202.1, 202.2, 202.3 Snap-on cups
- REF 204.1, 204.2, 204.3 Mounted on CA



### Manufacturer

Alfred Becht GmbH  
Carl-Zeiss-Str. 16,  
D-77656 Offenburg  
Tel: +49 781 60586-0  
Fax: +49 781 60586-40  
Germany  
[www.becht-online.de](http://www.becht-online.de)