

## Becht XL Wipes alkoholfrei

Ready-to-use disinfecting wipes

### Description

Recommended for disinfecting and cleansing of alcohol resistant medical devices, e.g. natural and synthetic leather of treatment chairs. Free of alcohol.

### Application

Place the bag in a dispenser bucket appropriate to fit the bag (e.g. Becht dispenser bucket, REF 531). Cut off bag below the welding seam. Use the first wipe to disinfect and clean the lid. Then take a single wipe from the middle of the roll and pull it through the opening of the lid. For the disinfecting and cleaning process wipe the surface thoroughly. Respect time of exposure. Then wipe surface with a dry and clean cloth. Close the bucket immediately after taking the wipe.

### Properties

- effective against bacteria (incl. MRSA) and yeast in 1 minute (EN 13727/13624)
- **successfully tested in the 4-field test acc. to EN 16615**
- inactivates HBV/HCV/HIV/BVDV/Vaccinia-viruses in 1 minute (EN 14476)
- specially soft, lint-free, absorbent and tear-resistant material
- free of alcohol and perfumes for best compatibility
- high volume of release of disinfecting solution on surface
- disinfecting liquid: 1,9 l Bechtosid soft
- gentle on materials
- size of wipes: 28 x 30 cm
- weight of wipes: 50 g/qm



### Composition

100 g of solution contain:

- 250 mg Benzyl-C12-16-alkyldimethyl-Chloride
- 250 mg Didecyldimethylammonium-Chloride
- 250 mg C12-14 Alkyl[(ethylphenyl) methyl]dimethyl-Chloride

### Form of delivery

- REF 531.90** Bag with 90 tissues
- REF 531** Dispenser bucket, reusable

### Shelf life

30 months

### Classification

Medical device, class IIa, rule 15,  
CE Certificate: G1 17 09 49628 010,  
TÜV SÜD Product Service GmbH, München

### Manufacturer

Alfred Becht GmbH  
Carl-Zeiss-Str. 16  
D-77656 Offenburg  
Tel.: +49 781 60586-0  
Fax.: +49 781 60586-40  
Germany  
[www.becht-online.de](http://www.becht-online.de)