

Bechtol plus ready-to-use solution

Description

The highly effective disinfecting and cleaning solution for rotating instruments

Application

Immediately after use place rotating instruments into Bechtol plus. Heavily soiled instruments should be soaked in warm water then cleaned with a soft brush. Ensure that the instruments are totally covered by the solution. Avoid the formation of air pockets. Respect times of immersion. Then remove from the solution, rinse thoroughly and dry with a clean cloth. Observe instructions of manufacturer of instruments.

Properties

- effective against bacteria (incl. MRSA) and yeast in 5 minutes
- kills spores in 60 minutes
- tested in accordance with the directives of the DGHM (Deutsche Gesellschaft für Hygiene und Mikrobiologie) of 01.09.2001
- inactivates HBV/HCV/HIV/HPV/Adeno/Polio viruses in 5 minutes (EN 14476)
- contains no phenols or aldehydes
- with fresh citrus scent
- biodegradable



Composition

100 g Bechtol plus contain:

- 0,5 g Potassium Hydroxide
- 0,1 g Natriumhypochlorite

Shelf life

30 months

Classification

Medical device, class IIb, rule 15,
CE Certificate: G1 17 09 49628 010,
TÜV SÜD Product Service GmbH, München

Form of delivery

REF 539 1L dispenser bottle, ready-to-use solution

REF 539.5 5L canister, ready-to-use solution

Accessories

REF 630 Bur sterilizer "Fräsator",
for HP mounted instruments
96/80 mm dia., 65 mm high



REF 631 Bur sterilizer "Fräsator",
for CA and FG mounted
instruments 78/60 mm dia.,
66 mm high

Manufacturer

Alfred Becht GmbH
Carl-Zeiss-Str. 16
D-77656 Offenburg
Tel.: +49 781 60586-0
Fax.: +49 781 60586-40
Germany
www.becht-online.de