

## Bechtid plus concentrate

### Description

The highly effective disinfecting and cleaning solution for use on all wet proofed surfaces, for ex. floors

### Application

Use Bechtid plus at the end of each working day. For measuring the quantity of concentrate press the bottle until the needed quantity has filled the dosage head. Dilute with tap water to obtain the ready-to-use solution. Wipe surfaces with a cloth soaked in Bechtid plus solution. Respect times of exposure.

### Properties

- effective against bacteria (incl. MRSA) and fungi in 15 minutes in a 2% solution (EN 1040/EN 1276)
- tested in accordance with the directives of the DGHM (Deutsche Gesellschaft für Hygiene und Mikrobiologie) of 01.09.2001
- inactivates HBV/HCV/HIV/Vaccinia/HPV/Adeno/Polio viruses in 15 minutes in a 5% solution
- disinfects, cleans and protects in just one application on all wet proofed surfaces



### Composition

- 100g Bechtid plus concentrate contain
- 6,9 g Didecyldimethylammoniumchlorid

### Shelf life

30 months

### Classification

Medical device, class IIa, rule 15,  
CE Certificate: G1 13 04 49628 007,  
TÜV SÜD Product Service GmbH, München

### Form of delivery

**REF 546** 1L dispenser bottle, concentrate, reusable

**REF 546.5** 5L canister, concentrate

### Accessories

**567.5** Dispenser pump for 5-Liter canister



### Expert opinions

Bactericidal and fungicidal in accordance with the directives of DGHM/VAH (Deutsche Gesellschaft für Hygiene und Mikrobiologie, revision 01.09.2001) for medical devices and acc. to EN standards

- Prof. Dr. med. H.-P. Werner / HygCen GmbH/Schwerin/ 2005
- Prof. Dr. med. Ralph Schubert/Frankfurt am Main/2002

### Manufacturer

Alfred Becht GmbH  
Carl-Zeiss-Str. 16  
D-77656 Offenburg  
Tel.: +49 781 60586-0  
Fax.: +49 781 60586-40  
Germany  
[www.becht-online.de](http://www.becht-online.de)

### Virucidal efficiency:

- Dr. Jochen Steinmann / Mikrolab GmbH / Bremen / 2002