

Prophy Brushes

Description

Tooth cleaning brushes are designed to remove dental plaque and to polish tooth surfaces using prophylactic paste. The purpose is to reach esthetic clean surfaces. Different shapes are available for specific applications (front/occlusal and interproximal polishing). All rotating brushes are mounted on a metal stem designed to fit contra-angle, screw-in and snap-on mounting.

Application

Only use prophylactic brushes together with a prophylactic paste. Place a small quantity of paste on the brush or directly on the tooth. Start polishing the tooth surface with rotating or oscillating movements. Soft brushes also allow cleaning beneath the gingival margin.

Properties

Observe an optimal motor speed of 2.500 rpm. Do not use pressure. All brushes are for single use and are supplied non-sterile. Non-autoclavable.

Shelf life

60 Months



Form of delivery

Cup Brushes, 4 mm, mounted on CA cups

- REF 260.21** with natural bristles, 12 pieces
- REF 260.230** with natural bristles, 100 pieces
- REF 260.2N** white, with nylon filament, 12 pieces
- REF 260.230N** white, with nylon filament, 100 pieces

- REF 260.240N** blue, with nylon filament, 100 pieces
- REF 260.260N** white soft, with nylon filament, 100 pieces
- REF 260.250N** pink soft, with nylon filament, 100 pieces

Classification

Medical Device, class IIa, rule 5
CE-Certificate: HD 60139053 0001,
TÜV Rheinland LGA Products GmbH, Nürnberg

Accessories

- REF 210** mandrel with screw shank to be mounted on angled handpiece
- REF 480** ProfiGuard RDA 250
- REF 481** ProfiPolish RDA 40
- REF 486** ProphyRepair for cleaning and polishing, final RDA 34



Manufacturer

Alfred Becht GmbH,
Carl-Zeiss-Str. 16,
D-77656 Offenburg,
Tel.: +49 781 60586-0,
Fax.: +49 781 60586-40,
www.becht-online.de