

ProfiPolish

Prophylactic paste

Description

Prophylactic paste for final polishing of vital teeth and fillings.

Application

After removal of dental calculus place ProfiPolish into a dappen dish or other suitable container. Use a small quantity on a rubber polisher or polishing brush and apply to the area of treatment. Only start handpiece if polisher is in contact with the tooth surface. Polish at low speed and with little pressure. Allow polisher to reach areas below the gingiva. Repeat the treatment with fresh paste until all surfaces are polished. Use Becht dental floss to apply the paste to the interproximal spaces. Finish by rinsing thoroughly

Properties

- Removal and polishing of plaque and discoloration
- Sodium-fluoride for effective protection against caries and parodontal diseases
- Mint flavour
- RDA 40
- 100% vegan, lactose-free
- gluten and phthalat-free

Composition

- Amorphous Siliciumdioxide
- Glycerine
- Paraben
- Sodium-fluoride (0,2%)
- flavor-agents



Form of delivery

REF 481 Tube with 95 g

Shelf life

60 months

Classification

Medical device, class I, Rule 5,

Accessories

Cup Brushes, 4 mm, mounted on CA cups:

- REF 260.21 white, with natural bristles
- REF 260.2N white, with nylon filament
- REF 260.240N blue, with nylon filament
- REF 260.250N soft, with nylon filament, pir
- REF 260.260N soft, with nylon filament, white



Prophylaxis Cups, latex-free

- REF 200.1, 200.2, 200.3 Screw-in cups
- REF 202.1, 202.2, 202.3 Snap-on cups
- REF 204.1, 204.2, 204.3 Mounted on CA



Manufacturer

Alfred Becht GmbH
Carl-Zeiss-Str. 16,
D-77656 Offenburg
Tel: +49 781 60586-0
Fax: +49 781 60586-40
Germany
www.becht-online.de