

Becht XL Wipes plus

Ready-to-use disinfecting wipes

Description

Recommended for disinfecting and cleaning of alcohol resistant medical surfaces, e.g. medical chairs, trays, units in dental and medical offices.

Application

Place the bag in a dispenser bucket appropriate to fit the bag (e.g. Becht dispenser bucket, REF 530). Cut off bag below the welding seam. Use the first wipe to disinfect and clean the lid. Then take a single wipe from the middle of the roll and pull it through the opening of the lid. For the disinfecting and cleaning process wipe the surface thoroughly. Respect time of exposure. Allow treated surface to dry without additional wiping. Close the bucket immediately after taking the wipe.

Properties

- effective against bacteria (incl. MRSA) and yeast in 1 minute
- **successfully tested in the 4-field test acc. to EN 16615**
- inactivates HBV/HCV/HIV/BVDV/Vaccinia-viruses (EN 14476) in 1 minute
- tuberculocidal (*M. terrae*) efficiency in 5 minutes (EN 14348)
- specially soft, lint-free, absorbent and tear-resistant material
- with fresh lemon scent
- high volume of release of disinfecting solution on surface
- recommended for alcohol resistant medical surfaces (Röhm test 12/2006)
- disinfecting liquid: 2,1 l Bechtozid plus
- size of tissue: 28 x 30 cm
- weight of wipes: 50 g/qm

Composition

100 g of solution contain:

- 250 mg Benzyl-C12-16-alkyldimethyl-Chloride
- 250 mg Didecyldimethylammonium-Chloride
- 250 mg C12-14 Alkyl[(ethylphenyl) methyl]dimethyl-Chloride
- 20 g ethanol
- 9,95 g 1-propanol

Form of delivery

- REF 530.90L Bag with 90 tissues
- REF 530 Dispenser bucket, reusable



Shelf life

30 months

Classification

Medical device, class IIa, rule 15,
CE Certificate: G1 17 09 49628 010,
TÜV SÜD Product Service GmbH, München

Manufacturer

Alfred Becht GmbH
Carl-Zeiss-Str. 16
D-77656 Offenburg
Tel.: +49 781 60586-0
Fax.: +49 781 60586-40
Germany
www.becht-online.de