

AquaHy concentrate

Description

AquaHy is a stabilizer liquid for spray devices used in podiatry. AquaHy reduces germs and prevents from building of a biofilm (algae forming). AquaHy is scent neutral and does not dehumidify the skin.

Application

The mixture can be used for at least 3 months. If increased light and temperature affects the spray fluid, a biofilm (slippery film in the tank) could grow in rare cases, although AquaHy was used. In this case do not use the spray fluid any longer and clean tank and spray system thoroughly. After use rinse system in contact with AquaHy with distilled water. Mixing ratio: 5l distilled water with 100 ml AquaHy. 1l distilled water with 20 ml AquaHy Never use tap water! Application concentration 1:50.

Properties

- reduces germs within a few minutes
- without scent
- easy to use
- economical (concentration 1:50)

Composition

100 g AquaHy contain:

- 0,5 g polymer hydrochloride

Form of delivery

REF 560 bottle with 100 ml concentrate

Classification

Medical device, class I, rule 1,



Shelf life

60 Months

Manufacturer

Alfred Becht GmbH
Carl-Zeiss-Str. 16
D-77656 Offenburg
Tel.: +49 781 60586-0
Fax.: +49 781 60586-40
Germany
www.becht-online.de