

Bechtozid Premium

Ready-to-use

Description

For the disinfection and cleaning of high-risk surfaces of medical devices, e.g. patient treatment chairs, unit handles and handpieces

Application

For use on medical devices, e. g. hand pieces and all alcohol resistant surfaces. Spray Bechtozid Premium on a clean cloth away from the body and avoid aspiration of the fog. Assure that the surface is covered by a complete layer. Respect time of exposure

Properties

- highly effective against bacteria (incl. MRSA) and yeast in 30 sec (EN 13727 / 13624)
- tuberculocidal (*M. terrae*) in 30 sec (EN 14348)
- inactivates HBV/HCV/HIV/HPV/BVDV/Vaccinia viruses in 30 sec (EN 14476)
- inactivates Adeno in 1 minute and Polio in 2 minutes (EN 13376)
- effective against H1N1-viruses (swine flu) in 30 sec (EN 14476)
- contains no phenols or aldehydes
- fast residue-free drying
- non-corrosive
- perfume-free or with fresh citrus scent
- pH-value: 11,5



Composition

100 g Bechtozid Premium contains:
60 g Ethanol

Shelf Life

30 months

Form of Delivery

- REF 561PF** 1-Liter bottle, ready-to-use solution, perfume-free, reusable, without spray head
- REF 561** 1-Liter bottle, ready-to-use solution, with citrus scent, reusable, without spray head
- REF 561.5PF** 5-Liter canister, ready-to-use solution, perfume-free
- REF 561.5** 5-Liter canister, ready-to-use solution, with citrus scent
- REF 561.10PF** 10-Liter canister, ready-to-use solution, perfume-free
- REF 561.10** 10-Liter canister, ready-to-use solution, with citrus scent

Accessories

- REF 567.5** Dispenser pump for 5-Liter canister
- REF 567.10** Dispenser pump for 10-Liter canister
- REF 544.SP** Spray head for 1-Liter bottle
- REF 567.51** Pouring spot for 5 and 10-Liter canister



Manufacturer

Alfred Becht GmbH
Carl-Zeiss-Str. 16
D-77656 Offenburg
Tel.: +49 781 60586-0
Fax: +49 781 60586-40
Germany
www.becht-online.de

Classification

Medical Device, Class IIa, Rule 15,
CE-Certificate: G1 17 09 49628 010,
TÜV SÜD Product Service GmbH, München