

## Becht Suction plus

### Concentrate

#### Description

Highly effective disinfecting, cleaning and deodorizing solution for dental aspirating systems with/without amalgam separator and spittoon bowls.

#### Application

To produce a 2% ready-to-use solution dilute 20 ml concentrate with 980 ml tap water. Then aspirate unit. Respect times of immersion and rinse system thoroughly with water. Repeat procedures at noon and in the end of working day to assure hygiene. Solution should not remain in the system during weekend. Use approx. 50 ml of the solution to disinfect and clean the spittoon bowl. Then rinse.

#### Properties

highly effective against bacteria (incl. MRSA) in 2%/5 minutes and yeast in 2%/30 minutes under high organic load (EN 13727/14562)

- inactivates Vaccinia-viruses, BVDV in 3%/15 minutes (EN 14476)
- tuberculocidal (*M.terrae*) with 2%/30 minutes under high organic load (EN 14348)
- no foaming solution
- contains no aldehydes, phenols or chlorides
- dissolves dirt from the tubes
- very gentle on materials



#### Composition

100 g of solution contain:

- 3,5 g Benzalkonium Chloride

#### Form of Delivery

REF 564 1-Liter bottle

REF 564.5 5-Liter canister

#### Shelf Life

30 months

#### Classification

Medical device, class IIa, rule 15,  
CE-Certificate: HD 60139053 0001,  
TÜV Rheinland LGA Products GmbH, Nürnberg

#### Accessories

REF 567.5 Dispenser pump for  
5 Liter canister



REF 567.51 Pouring spout for  
5 and 10 Liter canister



#### Manufacturer

Alfred Becht GmbH  
Carl-Zeiss-Str. 16  
D-77656 Offenburg  
Tel.: +49 781 60586-0  
Fax.: +49 781 60586-40  
Germany  
[www.becht-online.de](http://www.becht-online.de)